



SINGLE-TUBE RAD OPERATOR'S CONSOLE


Intelligen[®] has a new Operator's Console

Available on all INTELLIGEN or VALUEGEN[®] generators, this Operator's Console provides a full range of operational parameters for a busy x-ray department.


The console may be operated in two modes: **Manual Mode** and **AEC Mode**.

In the **Manual Mode** the operator selects kVp, mA, Time and mAs using the selection pushbuttons associated with each parameter.



In the **AEC Mode** the operator must ensure that a sufficient backup time is selected to provide for an automatic exposure termination. The AEC Mode is also dependent upon having the optional AEC module installed.

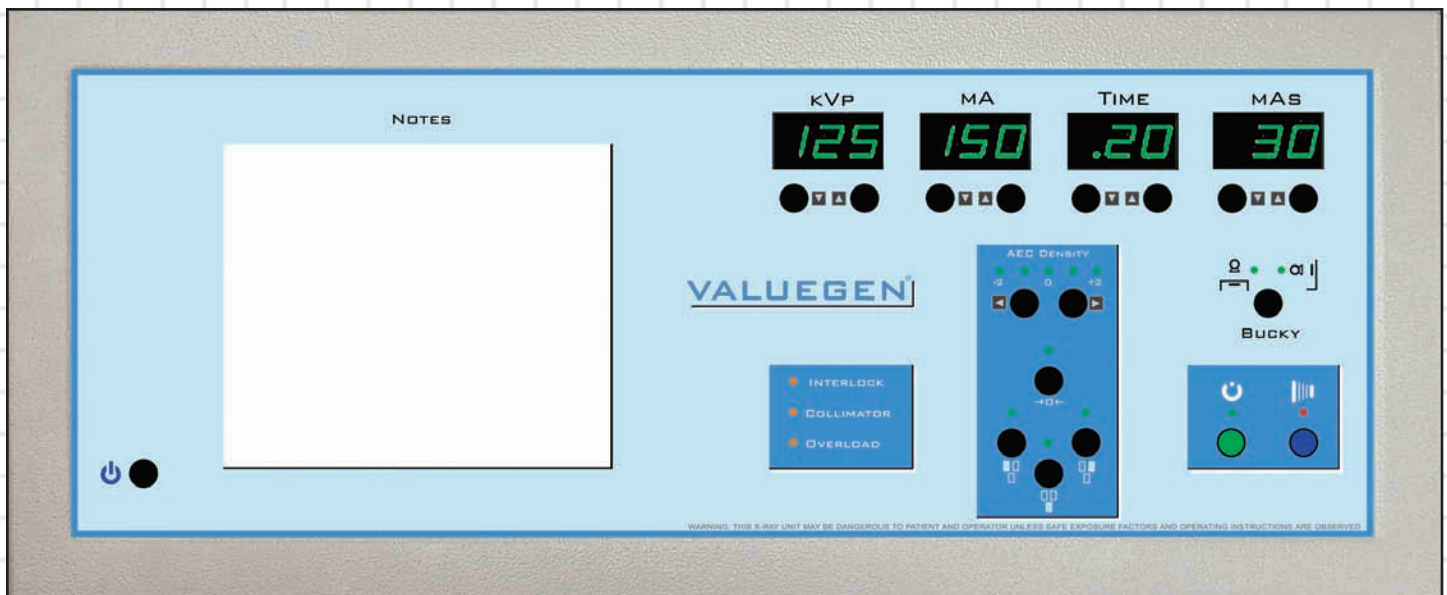

SINGLE TUBE RAD OPERATOR'S CONSOLE

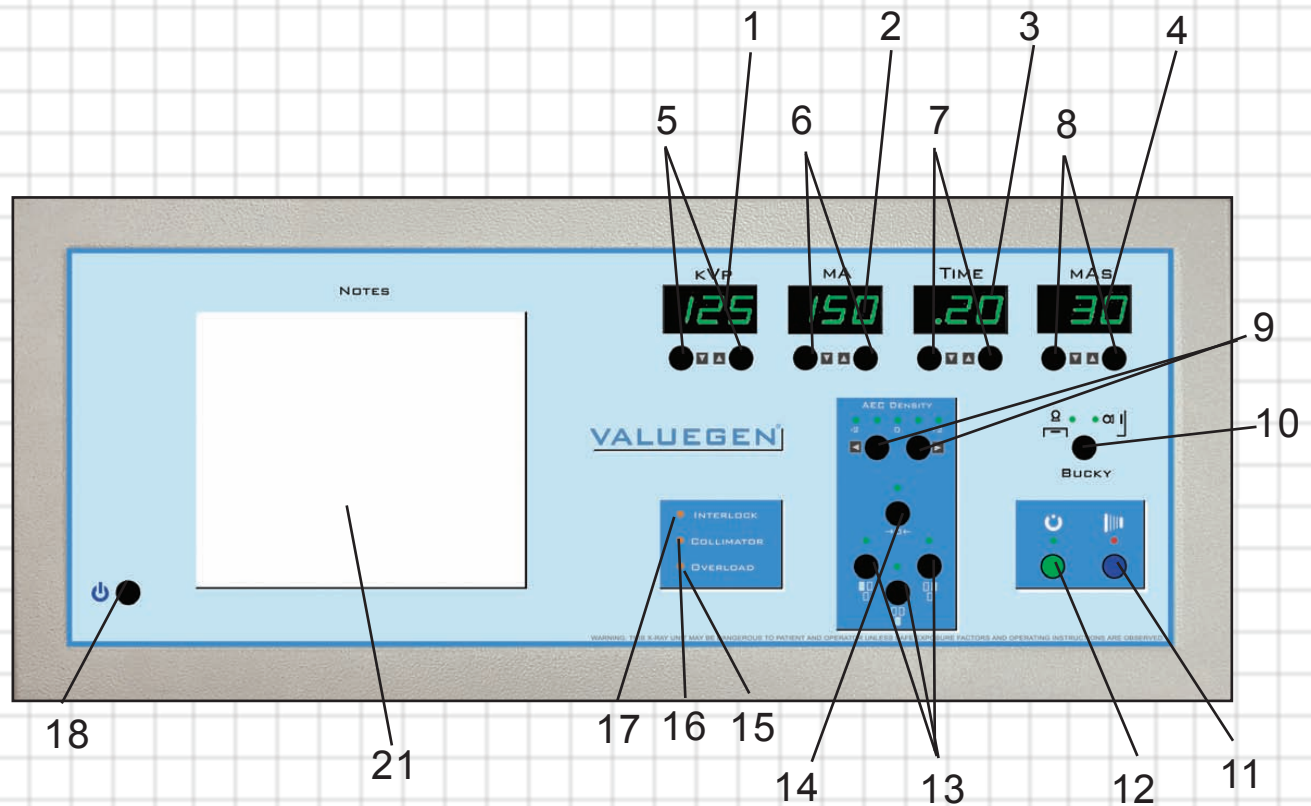

Single Tube Rad Console
w/ optional stand

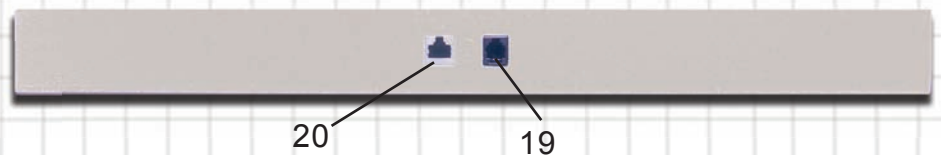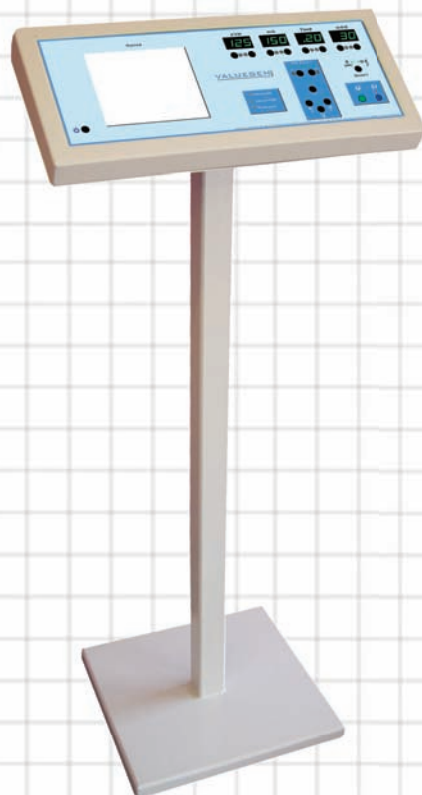

- | | |
|------------------------|----------------------|
| 1. kVp Display | 12. PREP |
| 2. mA Display | 13. AEC Field Select |
| 3. Time Display | 14. AEC Reset |
| 4. mAs Display | 15. Overload LED |
| 5. kVp Up/Down | 16. Collimator LED |
| 6. mA Up/Down | 17. Interlock LED |
| 7. Time Up/Down | 18. On/Off |
| 8. mAs Up/Down | 19. Hand Switch |
| 9. AEC Density Up/Down | 20. Connector |
| 10. BKY Select | 21. White Board |
| 11. EXP | |

Console Size: 8.5"x20"x2"

GTR[®] Labs, Inc.

510 Elk St.
Gassaway, WV 26624

Phone: 304-364-2211
Fax: 304-364-2212

gtr@gtrllc.com
www.gtrllc.com

